medical biomaterial products

# MB-Collagen

- GB** Collagen dressing for wound care  
Instructions for use
- DE** Kollagenvlies – Wundauflage  
Gebrauchsanweisung
- FR** Tampon de collagène pour le traitement des plaies  
Notice d'utilisation
- ES** Relleno de colágeno para el cuidado de heridas  
Información de uso
- IT** Rivestimento di collagene per bendaggio chirurgico  
Istruzioni per l'uso
- PT** Curativo de colágeno para o tratamento de feridas  
Instruções de uso
- TR** Kollajen dokumasız kumaş – Yara örtüsü  
Kullanım talimatı
- EL** Φλινς κολλαγόνου – επίδεσμος τραύματος  
Οδηγίες χρήσης

CE0483

REF	PZN	Size/ Größe/ Dimension/ talla/ dimensione/ tamanho/ boyut/ μέγεθος	UDI	Content/ Inhalt/ Contenu/ Contenido/ Contenuto/ Conteúdo/ İçerik/ Περιεχόμενο Box
0505C	6631492	50*50 mm	0426023091001	3
0510C	6631463	50*100 mm	0426023091002	3
1010C	6631486	100*100 mm	0426023091003	3

In the event of a serious incident involving MB-Collagen, please report it to MBP Medical Biomaterial Products GmbH and to the competent authority in your Member State.

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URL: <http://www.mbp-gmbh.de/mb-collagen>

GB

## **MB-Collagen, Collagen dressing for wound care**

sterile, for single use

### **Composition:**

sterile porcine collagen matrix

### **Mode of action**

The porous structure of the collagen sponge provides a capillary suction effect and thus absorption of the wound secretion. The wound secretion dissolves the porous structure and releases the native collagen. This has an accelerating effect on the formation of new granulation tissue in the granulation phase. The added collagen indirectly influences the epithelialization or regeneration phase by stimulating the body's own collagen synthesis, whereby epithelialization progresses rapidly. The newly formed tissue offers improved starting conditions for good cosmetic results, as is known from animal and human clinical studies.

### **Indications**

Collagen dressing for wound care, single use, for

- non-infected wounds with wound healing disorders
- Wounds with secondary wound healing in the granulation phase

- Wounds in the epithelialization phase (e.g., leg ulcers, decubital ulcers, split-thickness wounds, burn wounds, wounds with secondary healing, etc.).

### **Contraindications**

Use on infected wounds.

### **Side effects**

In individual cases, intolerance reactions to collagen occur. Occasionally, pain occurs after application of a dry preparation to the wound surfaces. Very rarely, existing infections are aggravated.

### **Use during Pregnancy and lactation**

No studies are available on the use of MB-Collagen during pregnancy and breast-feeding or on its effect on human fertility. Before MB-Collagen is used, the attending physician must therefore weigh up the benefits for the mother and the possible risks for the child on an individual basis.

### **Interactions**

Antiseptics that release chlorine (e.g. chloramine), as well as tannin and caustics that alter proteins, must not be used together with collagen. Ointments and powders as well as silicone preparations can alter the interstices of collagen films, fleeces or sponges, so they should

not be applied together with collagen. Do not fix with polymethyl methacrylate adhesives. Protein-damaging substances (tannic acid, silver nitrate) should also not be used. Wound disinfections should not be combined with MB-Collagen in the form of a moist wound dressing.

### **Dosage and method of use**

Unless otherwise prescribed by the doctor, a collagen sponge corresponding to the size of the wound is placed on the moist wound under aseptic conditions. Fixation can be done with a non-woven compress or a gauze compress. The frequency of dressing changes depends on the amount of wound secretion:

- In case of heavy secretion: several times a day
- With moderate secretion: 1 x daily
- In case of weak exudation: after several days (provided that no signs of inflammation are present)

In case of very weak or absent exudation, it is necessary to moisten MB-Collagen with a physiological sodium chloride or Ringer's solution.

### **Storage instructions**

The unopened, sterile pack must be stored below 24°C in a cool and dry place with good ventilation. MB-Collagen must be kept away from extremes of temperature

and humidity. Do not use MB-Collagen after the expiry date.

### **Warnings, to be observed**

MB-Collagen is sterilized with  $\gamma$ -radiation and must not be resterilized. Sterility is only guaranteed if the packaging is undamaged. MB-Collagen must not be reused once it has been removed from the packaging and/or has come into contact with a patient, as there is then an increased risk of contamination with subsequent risk of infection.

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