medical biomaterial products

Diacoll

GB Hemostatic agent for topical use
Instructions for use
DE Haemostyptikum zur äußeren Anwendung
Gebrauchsanweisung
FR Agent hémostatique pour application topique
Notice d'utilisation
ES Agente hemostático para uso tópico
Información de uso
IT Antiemorragico per uso esterno
Istruzioni per l'uso
PT Agente hemostático para uso tópico
Informação de uso
TR Resorbe edilebilir, lokal hemostatik / harici kullanım için Kullanım talimatı
EL Απορροφήσιμο αιμοστατικό για τοπική και εξωτερική χρήση Οδηγίες χρήσης

CE0483

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Sterile porcine collagen matrix

Interaction with other agents

Antiseptics, which release chlorine (e.g. chloramine) as well tannic acid and caustics which change the protein must not be used together with collagen. Ointments, powder as well silicon preparations can change intercostal space at collagenfoil, -sponge and -fleece, and should not be used together with collagen.

Indication areas

DIACOLL is indicated wherever capillary, venous, small arterial and diffuse seeping bleedings must be stopped, and where conventional means of hemorrhage control are either inadequate or technically difficult and time-consuming.

Indications

General Surgery

Vascular, Cardiothoracic Surgery

Neurosurgery

Maxillofacial Surgery

ENT

Urology and Gynecology

Dosage and application

After opening the package, DIACOLL is taken out with dry sterile instruments, placed on the cleaned sterile wound and pressed on lightly with appropriate resources (e.g. swabs). DIACOLL can be cut to the necessary size with sterile scissors. Immediately after contact with the wet wound surface DIACOLL sticks to the wound and forms a gel like bond with blood. Unnecessary collagen material should be removed. With superficial wounds an additional layer of cotton gauze fixed with elastic bandages is recommended.

Contraindications

DIACOLL is contra-indicated in infected wounds, and must not be used concurrently with methyl methacrylate adhesive; it must not be used for intravasal application.

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