# Hemocol

GB Absorbable, local hemostyptic agent

Instructions for use

DE Resorbierbares, lokales Haemostyptikum

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FR Hemostatique local, resorbable

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ES Hemostático local, reabsorbible

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IT Emostatico locale, assorbibile

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PT Hemostático local, reabsorvel

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Sterile porcine collagen matrix

## Interaction with other agents

Antiseptics, which release chlorine (e.g. chloramine) as well tannic acid and caustics which change the protein must not be used together with collagen. Ointments, powder as well silicon preparations can change intercostal space at colla-genfoil, -sponge and -fleece, and should not be used together with collagen.

## Indication areas

HEMOCOL is indicated wherever capillary, venous, small arterial and diffuse seeping bleedings must be stopped, and where conventional means of hemorrhage control are either inadequate or technically difficult and time-consuming.

## Indications

General Surgery  
Vascular, Cardiothoracic Surgery  
Neurosurgery  
Maxillofacial Surgery  
ENT  
Urology and Gynecology

## Dosage and application

After opening the package, HEMOCOL is taken out with dry sterile instruments, placed on the cleaned sterile wound and pressed on lightly with appropriate resources (e.g. swabs). HEMOCOL can be cut to the necessary size with sterile scissors. Immediately after contact with the wet wound surface HEMOCOL sticks to the wound and forms a gel like bond with blood. Unnecessary collagen material should be removed. With superficial wounds an additional layer of cotton gauze fixed with elastic bandages is recommended.

# Hemocol

GB

Absorbable, local hemostyptic agent

---

## Contraindications

HEMOCOL is contra-indicated in infected wounds, and must not be used concurrently with methyl methacrylate adhesive; it must not be used for intravasal application. If you use an absorbable hemostatic agent on or near bony or neural spaces:

- use the minimum amount necessary to achieve hemostasis; and,
- remove as much of the agent as possible after hemostasis is achieved.

This will reduce the likelihood of neural and other soft tissue damage from swelling of the absorbable hemostatic agent, and/or migration and swelling of fragments of the agent.

## Side effects

Allergic reactions may occur in isolated cases. Sometimes pain appears after application of a dry preparation to the wound surface. The increase of existing infection is very rare.

## Use during pregnancy and breastfeeding

There has neither been any testing on the use of HEMOCOL during pregnancy and breastfeeding nor on its impact on human fertility. Therefore the advantages for the mother and the possible risks for the child must be assessed individually before using HEMOCOL.

# Hemocol

Absorbable, local hemostyptic agent

---

## Note!

### HEMOCOL

- is sterilized by  $\gamma$ -irradiation and must not be resterilized
- must be considered unsterile and must be discarded if the package is damaged
- must not be used after the expiry date
- must not be stored above 24°C
- must be protected against moisture
- is a medical device and therefore must be kept from children!

After it has been removed from the packaging and/or has come in contact with a patient, the HEMOCOL collagen fleece cannot be reused without increased risk of contamination and subsequent infection.

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MBP Medical Biomaterial Products GmbH / Lederstraße 7 /